GB PLAQUE DISCLOSING SYSTEM

A bright new light in Plaque Disclosants

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GB PLAK-CHECK™ Plaque Disclosing System uses a sodium fluorescein liquid and a special blue light to disclose plaque on the tooth surface to help improve brushing and flossing for better oral care.

Composition: Sodium Fluorescein 0.75%

Contraindications: Patients with a known allergy to sodium fluorescein food dye.

Using Plak-Check™ System: With the swab still inside the cellophane, hold swab with the colored ring end up. Bend head of applicator stick until it snaps. Do not break off completely. Liquid will flow into the cotton head at the other end of the swab within a few seconds. Split the cellophane wrapper at the middle of both sides of the swab and pull the cellophane from the fluorescein soaked end. This will prevent the liquid from getting on your hands. Rub the cotton head soaked with the Plak-Check™ disclosing solutions below the tongue behind the lower front teeth for 4-5 seconds. Have the patient swish the liquid around the mouth for five seconds to coat all teeth. Rinse the mouth with a small amount of water. To visualize the plaque, have the patient hold the light unit in front of his or her face so the teeth can be seen in the mirror. Turn on the special disclosing light by depressing and holding the button on the front of the unit. Plaque will glow yellow when exposed to this light. A small amount of yellow color may be visible on the teeth and tongue under normal light. If desired, have the patient brush to remove remaining color.

Caution: Keep unused swabs in carton and away from light.

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